Resin-based Dental Restorative Material

PALFIQUE UNIVERSAL FLOW**ENGLISH**

Carefully read all information, precautions, notes and materials before using PALFIQUE UNIVERSAL FLOW.

■ PRODUCT DESCRIPTION AND GENERAL INFORMATION


- 1) PALFIQUE UNIVERSAL FLOW is a low viscosity, light-cured, radiopaque composite resin for use in anterior and posterior restorations. Direct placement of PALFIQUE UNIVERSAL FLOW into a prepared cavity allows for easy handling.
- 2) PALFIQUE UNIVERSAL FLOW incorporates Radical-Amplified Photopolymerization initiator technology (RAP technology). RAP technology facilitates a shortened light curing time and ample working time. Please see the table depicting the relationship between curing time and increment depth (■ INDICATIONS FOR FILLING AND CURING).
- 3) PALFIQUE UNIVERSAL FLOW comes in three viscosities, High, Medium and Super Low. Each type contains spherical silica-zirconia filler and composite filler as indicated in below table. The composite filler offers reduced polymerization shrinkage stress due to its round shape. The spherical filler contained in PALFIQUE UNIVERSAL FLOW is not indicated for direct pulp capping.

6. Bonding System
Apply a bonding system according to its manufacturer's instructions.

- In case of porcelain/composite repairs be sure to condition the surface of porcelain/composite with a silane coupling agent BEFORE performing the bonding procedure.

7. Dispensing**7-1. L-SYRINGE (High, Medium and Super Low)**


- Hold the nozzle and turn the cap counterclockwise for removal. [1]
- Hold the nozzle and place the Dispensing Tip (metal tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]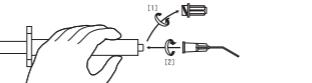

- Accessories and Dispensing Tip are exclusively designed for the L-SYRINGE of PALFIQUE UNIVERSAL FLOW. Use the attached Dispensing Tips (metal tips).
- Dispensing Tips (TOKUYAMA Dispensing Tip) are also sold separately.
- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

7-2. SYRINGE (Medium)

- Hold the nozzle and turn the cap counterclockwise for removal. [1]
- Hold the nozzle and place the Dispensing Tip (plastic tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

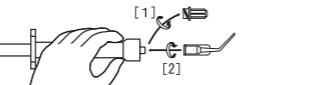- Accessories and Dispensing Tip are exclusively designed for the SYRINGE of PALFIQUE UNIVERSAL FLOW. Use the attached Dispensing Tips (plastic tips).
- Dispensing Tips (TOKUYAMA Plastic Tip) are also sold separately.
- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

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■ CLINICAL PROCEDURE**1. Cleaning**

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

Prior to isolation of tooth, select the appropriate shade of PALFIQUE UNIVERSAL FLOW using shade guide provided for PALFIQUE UNIVERSAL FLOW.

3. Isolation

A rubber dam is the preferred method of isolation.

4. Cavity Preparation

Prepare the cavity. In case where no cavity preparation has been made (caries-free cervical defects), clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water.

5. Pulp Protection

Calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL-BASED MATERIALS to protect the pulp as these materials will inhibit curing PALFIQUE UNIVERSAL FLOW. PALFIQUE UNIVERSAL FLOW is not indicated for direct pulp capping.

6. Bonding System

Apply a bonding system according to its manufacturer's instructions.

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- Accessories and Dispensing Tip are exclusively designed for the L-SYRINGE of PALFIQUE UNIVERSAL FLOW. Use the attached Dispensing Tips (metal tips).

- Dispensing Tips (TOKUYAMA Dispensing Tip) are also sold separately.

- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

7-2. SYRINGE (Medium)

- Hold the nozzle and turn the cap counterclockwise for removal. [1]

- Hold the nozzle and place the Dispensing Tip (plastic tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

- Accessories and Dispensing Tip are exclusively designed for the SYRINGE of PALFIQUE UNIVERSAL FLOW. Use the attached Dispensing Tips (plastic tips).

- Dispensing Tips (TOKUYAMA Plastic Tip) are also sold separately.

- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

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